BENEFIT[®]

Leading in Orthodontic Implants
psm-medical.com

Quality is never an accident.

It is always the result of high intention, sincere effort, intelligent direction and skillful execution; it represents the wise choice of many alternatives.

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Introduction | Einführung

Skeletal anchorage in orthodontics

An important requirement for successful orthodontic treatment is stable anchorage.

Conventional orthodontic anchorage can be limited by periodontal disease and/or missing teeth. Skeletal anchorage has been proven to be very effective. It can be an essential part of non compliance therapy and broadens the options available to the orthodontic practitioner.

Mini Implants are especially advantageous because of the minimally invasive insertion protocol and their relatively low cost.

However, limitations can occur when conventional Mini Implant Systems are used for orthodontic anchorage. In collaboration with leading specialists, **PSM** has developed the unique **BENEFIT®** TAD System for temporary fixation in the palate. **BENEFIT®** offers high precision titanium implant screws and a large variety of prefabricated connecting parts such as **BENEplates®**, abutments and palatal expanders.

With the introduction of the **BENEFIT®** System **PSM** has considerably enhanced the therapeutic diversity of the treatments available for Mesialization, Distalization, RPE as well as uprighting or anchorage of the dentition.

With **BENEFIT®**, **PSM** was already in 2007 the first company to offer a full range of skeletal anchorage appliances for the palate – a trend that has become „state of the art“ in Orthodontics.

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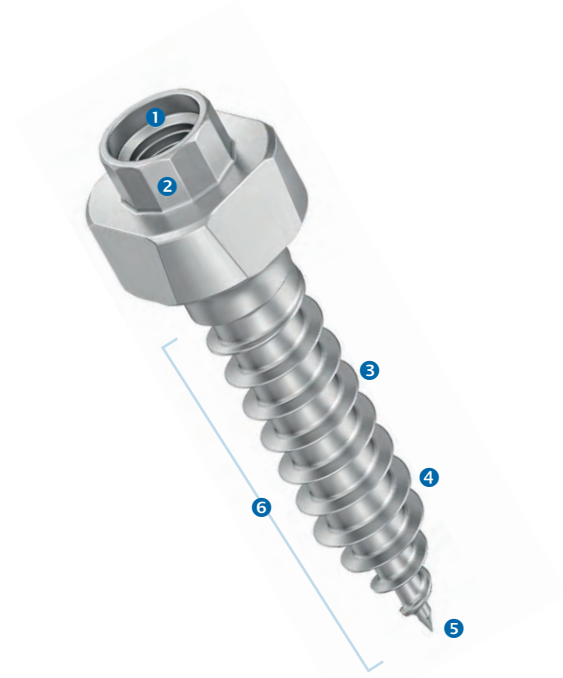
BENEFIT® Screw System | **BENEFIT®** Schrauben System

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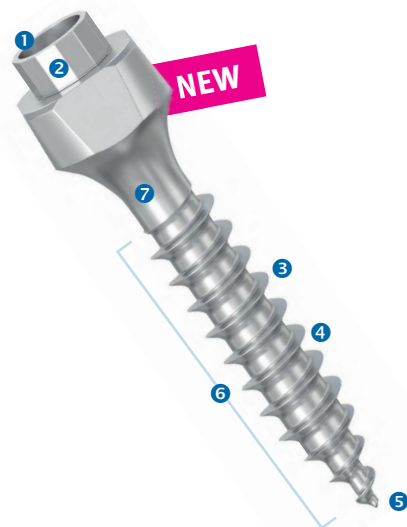
Developed in cooperation with: | Entwickelt in Zusammenarbeit mit:
Dr. Renzo De Gabriele, Fazzi Hospital Private Practice, Lecce, (LE) Italy

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BENEFIT® Screws | BENEFIT® Schrauben

BENEFIT® Orthodontic Screws, Titanium

BENEFIT® KFO-Schrauben, Titan

ST P1

L1	Ø 1.5 mm	Ø 1.8 mm	Ø 2.0 mm	Ø 2.3 mm	
5 mm	–	–	ST-33-54205 NEW	–	
7 mm	ST-33-54107	ST-33-54187	ST-33-54207	ST-33-54307	
8 mm	–	–	ST-33-54208 NEW	–	
9 mm	ST-33-54109	ST-33-54189	ST-33-54209	ST-33-54309	
10 mm	–	–	ST-33-54210 NEW	–	
11 mm	ST-33-54111	ST-33-54191	ST-33-54211	ST-33-54311	
13 mm	–	–	ST-33-54213	ST-33-54313	
15 mm	–	–	ST-33-54215	ST-33-54315	

M 3:1

BENEFIT® Orthodontic Screws, Titanium

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
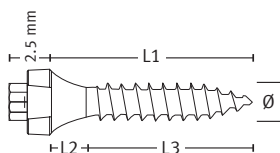
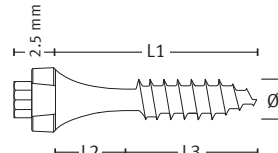
M 3:1

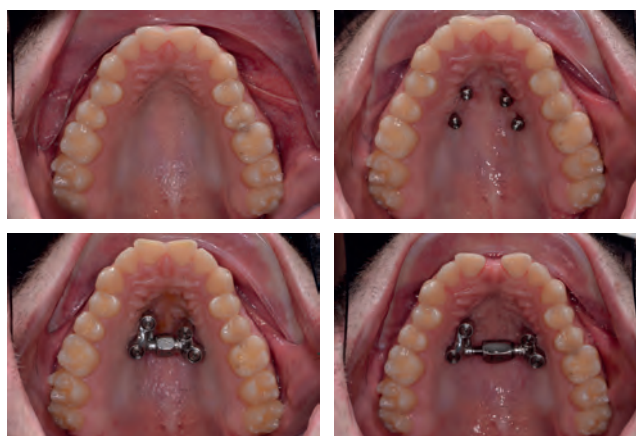Developed in cooperation with:
 Entwickelt in Zusammenarbeit mit:

 Dr. Renzo De Gabriele,
 Fazzi Hospital Private Practice
 Lecce, (LE) Italy

BENEplates median | BENEplates median

BENEplates, L/M/S/XS, incl. 2 fixation screws, stainless steel

BENEplates, L/M/S/XS, inkl. 2 Fixierschrauben, Implantatstahl

Without wire Ohne Draht				
L MAX 14.7 mm MIN 6.7 mm 17.5 mm 33-54400 P1	M NEW MAX 11.6 mm MIN 5.2 mm 15.3 mm 33-54414 P1	S MAX 9.1 mm MIN 3.7 mm 12.5 mm 33-54402 P1	XS MAX 6.3 mm MIN 3.7 mm 10.8 mm 33-54424 P1	
With Ø 1.1 mm wire (12 cm) Mit Ø 1,1 mm Draht (12 cm)				
L 1.1 mm 33-54429 P1	M NEW 1.1 mm 33-54417 P1	S 1.1 mm 33-54409 P1	XS 1.1 mm 33-54423 P1	
With Ø 0.8 mm wire (12 cm) Mit Ø 0,8 mm Draht (12 cm)		With Ø 1.1 mm & 0.8 mm wire (12 cm) Mit Ø 1,1 mm Draht & 0,8 mm Draht (12 cm)		
L 0.8 mm 33-54428 P1	S 0.8 mm 33-54408 P1	S 0.8 mm 1.1 mm 33-54475 P1		

BENEplates, S, incl. 2 fixation screws, **Titanium/TMA**

BENEplates, S, inkl. 2 Fixierschrauben, **Titan/TMA**

Without wire Ohne Draht		
S 1.1 mm 33-54409-TI P1	S 0.8 mm 33-54420 P1	L 33-54407 P1

BENEplate, L, with bracket,
incl. 2 fixation screws,

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BENEplates | BENEplates


BENEplates, L/S, incl. 2 fixation screws, stainless steel

BENEplates, L/S, inkl. 2 Fixierschrauben, Implantatstahl

BENEplate, L , median, with central 1.1 mm wire (120 mm) BENEplate, L , median, mit 1,1 mm Draht (120 mm)		P1
L		1.1 mm
	acc. nach Dr. Nienkemper, Düsseldorf, Germany	
33-54480		

BENEplate, L , T-Mesialslider plate, median, with central 1.1 mm wire + 0.8 mm wire anteriorly (120 mm) BENEplate, L , T-Mesialslider plate, median, mit 1,1 mm mittigem Draht + 0,8 mm Draht anterior (120 mm)		P1
L		0.8 mm
	1.1 mm	
33-54485		

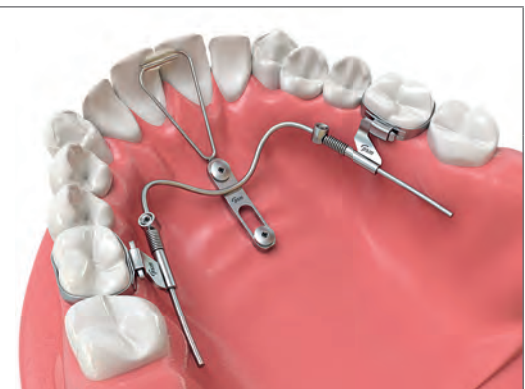BENEplate, L , T-Mesialslider plate, median, with central 1.1 mm wire (120 mm) + 0.8 mm wires lateral (60 mm/ea.) BENEplate, L , T-Mesialslider plate, median, mit 1,1 mm mittigem Draht (120 mm) + 0,8 mm Drähte lateral (je 60 mm/Stk.)		P1
L		0.8 mm
	60 mm	
33-54490		

BENEplate, L , T-Mesialslider plate, paramedian, with 1.1 mm lateral wire (120 mm) + 0.8 mm wire vertically BENEplate, L , T-Mesialslider plate, paramedian, mit 1,1 mm Draht (120 mm) lateral + 0,8 mm Draht vertikal		P1
L		0.8 mm
	55 mm	
33-54492		

BENEplate, S , T-Mesialslider plate, paramedian, with 1.1 mm lateral wire (120 mm) + 0.8 mm wire vertically BENEplate, S , T-Mesialslider plate, paramedian, mit 1,1 mm Draht (120 mm) lateral + 0,8 mm Draht vertikal		P1
S		0.8 mm
	55 mm	
33-54493		



 Video

PSM –
BENEFIT® Arnoudizer

BENEplate | BENEplate

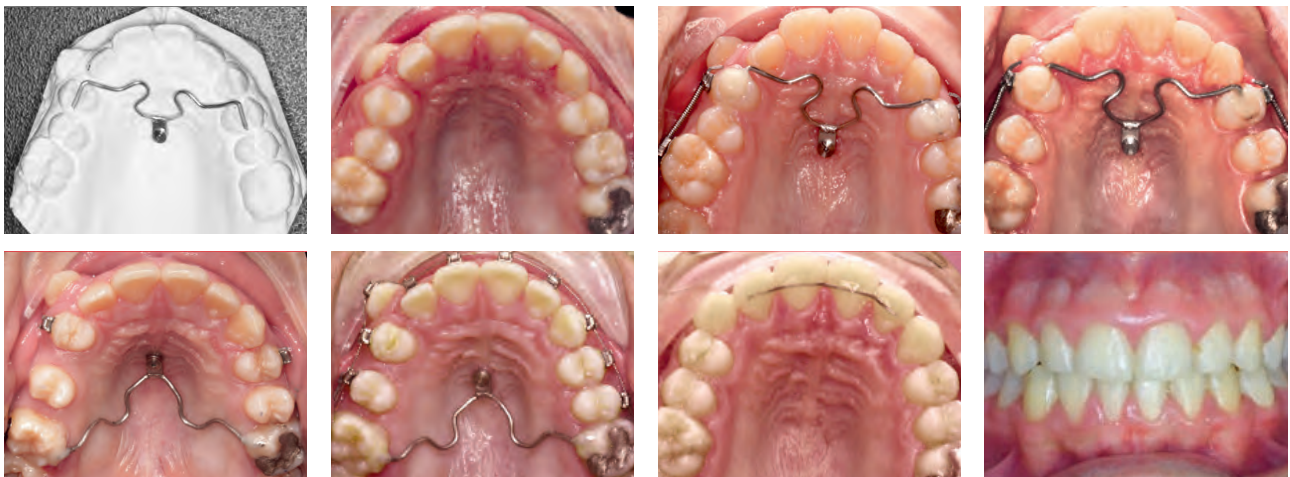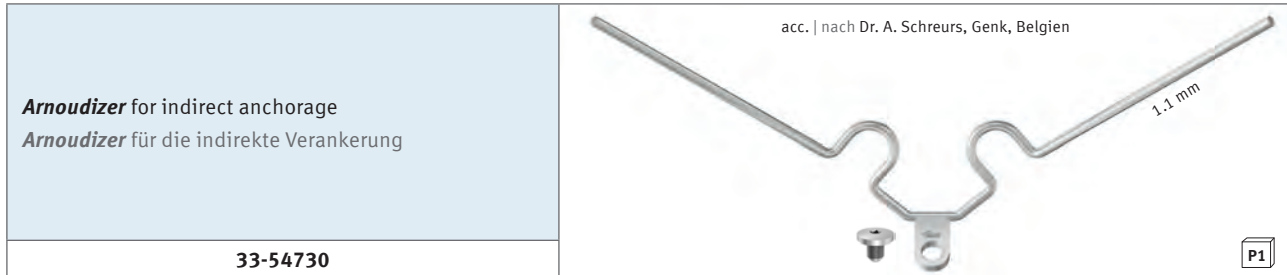Arnoudizer, incl. 1 fixation screw, stainless steel
Arnoudizer, inkl. 1 Fixierschraube, Implantatstahl

A device for molar distalization. The upper molars will be derotated, uprighted and distalized in only two or three months. By converting the system in a transpalatal bar, it can be used as an anchor for the molars. The anterior teeth can be retracted and aligned simply.

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

BENEsider | BENEsider – DISTALIZATION

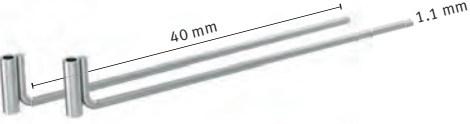Micro Mobilizer , for wire 0.5 mm to 1.2 mm Micro Mobilisator , für 0,5 mm bis 1,2 mm Draht P2		Mobilizer , for wire 0.5 mm to 1.2 mm Mobilisator , für 0,5 mm bis 1,2 mm Draht P2	
33-54543		33-54540	
BENEtube, S , 4.5 mm BENEtube, S , 4,5 mm P2		BENEtube, extended BENEtube, verlängert acc. nach to Dr. S. Yamaguchi P2	
33-54535		33-54558	
BENEtube, M , 6.5 mm BENEtube, M , 6,5 mm P2			
33-54536			
Flexitube, long Flexitube, lang acc. nach to Dr. M. Schubert P2		BENEsider, NiTi springs BENEsider, NiTi Federn	
33-54549		33-54524	240 g
Flexitube, short Flexitube, kurz P2		33-54525	500 g
33-54550			
BENEtube, with wire 1.1 mm, 40 mm BENEtube, mit Draht 1,1 mm, 40 mm acc. nach to Dr. T. Banach P2			
33-54544			

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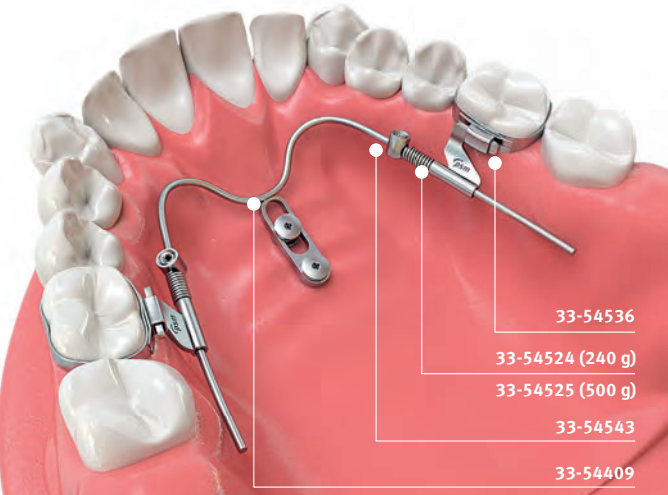The BENEsider

The most classical indication for skeletal anchorage using Mini-Implants with abutments is molar-distalization and/or mesialization in the maxilla. By means of the so called "**BENEsider**" the molars can be bodily distalized and/or mesialized using the Mesialslider. In many cases, tooth extraction or dental implants can be avoided. To couple two **BENEFIT®** Mini-Implants, a **BENEplate** with prewelded wire is connected to the implants and the **BENEsider** parts (mobilizer, springs, **BENETube**) are placed on the wire. Active force is applied by springs (240 g or 500 g) and activated with the Mobilizer.

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
Abutments | Abutments

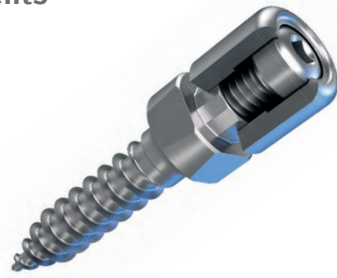Abutments, stainless steel
Abutments, Implantatstahl

Abutment, Standard Abutment, Standard	P1	
33-54430		
Abutment, Standard with 1.1 mm wire (12 cm) Abutment,, Standard mit 1,1 mm Draht (12 cm)	P1	
33-54460		

Sleeves for BENEFIT® screws, incl. fixation screws Hülsen für BENEFIT®-Schrauben, inkl. Fixierschrauben	P2	
33-57620		

Abutment, with loop Abutment, mit Öse	P1	
33-54445		
Abutment, with 1 bracket Abutment, mit 1 Bracket	P1	
33-54450		
Abutment, with 2 brackets Abutment, mit 2 Brackets	P1	
33-54452		

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
PSM –
BENEFIT® Temporary Pontic
with PEEK Abutment


BENEFIT® PEEK Abutment, incl. 1 fixation screw BENEFIT® PEEK Abutment, inkl. 1 Fixierschraube	P1	
33-54466		

BENEFIT® PEEK Abutment, 15° angled, incl. 1 fixation screw BENEFIT® PEEK Abutment, 15° abgewinkelt, inkl. 1 Fixierschraube	P1	
33-54469		

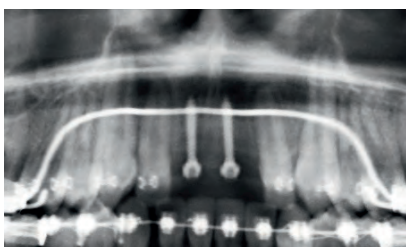
bm^x® – BENEFIT® Maxillary Xpander | bm^x® – BENEFIT® Maxillary Xpander

10 mm max. Expansion	
bm ^x ® palatal Expander, with 2 fixation screws bm ^x ® Gaumennahtspreizer, mit 2 Fixierschrauben	
10 mm max. expansion 10 mm max. Ausdehnung	
6 mm	33-57506
8 mm	33-57508
10 mm	33-57510

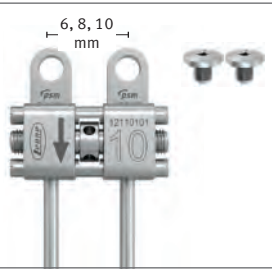After using the **BENEFIT®** Screw for the **Hybrid Hyrax®** appliance very successfully in the last years the **bm^x®** now follows the Orthodontists request for an entirely **BENEFIT®** mini-implant anchored appliance for Rapid maxillary expansion on young patients. Using the De Franco Screwguide (Page 36) the distance and the parallelity of the **BENEFIT®** Screws can be exactly determined, directly on the patient. An impression is no longer necessary. Alternatively visualisation can of course be made by means of digital imaging or traditional impression taking.

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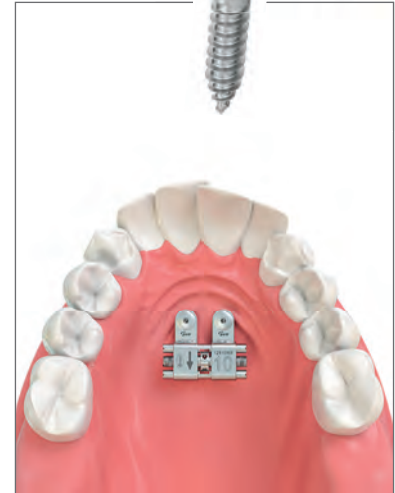Hybrid Hyrax® RPE | Hybrid Hyrax® GNE

The **Hyrax®** Ring and **Hyrax®** Abutment facilitate the construction and fixation of a **Hybrid Hyrax®** appliance. It allows welding or soldering with the **Hyrax®** Screw as well as little rotation movements when connecting the appliance to the **BENEFIT®** Screws. The **Hyrax®** Ring is supplied with 1 fixation screw each.

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

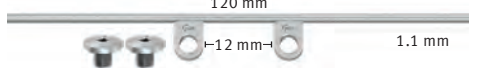
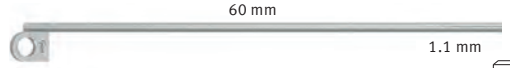
Mouse Trap – for Molar Intrusion | Mausefalle – zur Molarenintrusion

Recommendation | Vorschlag



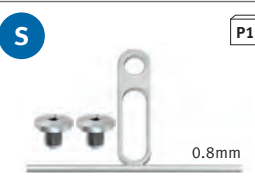
median median	BENEplate S , median, with 0.8 mm wire (12 cm), incl. 2 fixation screws		P1
	BENEplate S , median, mit 0,8 mm Draht (12 cm), inkl. 2 Fixierschrauben		
	33-54408		
paramedian paramedian	BENEplate S , paramedian, with 0.8 mm wire (12 cm), incl. 2 fixation screws		P1
	BENEplate S , paramedian, mit 0,8 mm Draht (12 cm), inkl. 2 Fixierschrauben		
	33-54474		
	BENEsli der hook BENEsli der Haken		P10
	33-54523		

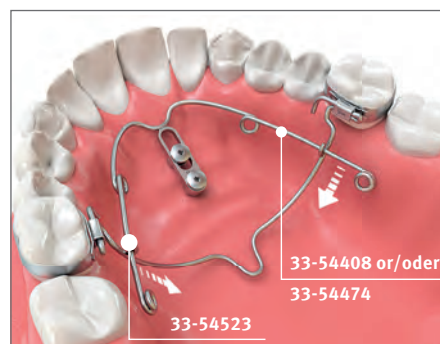BENEFIT® Orthodontic Screw, Ø 2.0 x 9.0 mm
BENEFIT® KFO-Schraube, Ø 2,0 x 9.0 mm

ST-33-54209-2 (9 mm)

Mobilholder | Mobilholder

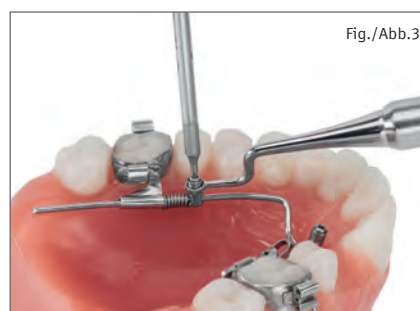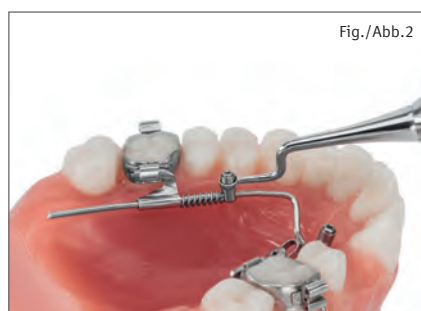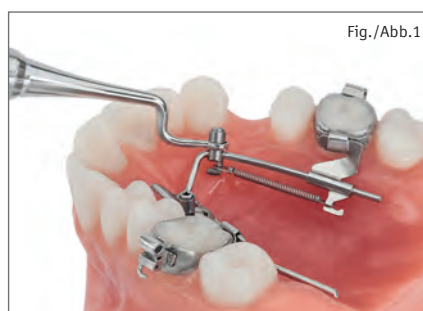The new Mobilholder is intended to move the **BENEFIT®** mobilizers into the final position prior to tightening the set screw for distalization (fig. 2, 3) or mesialization (fig. 1).

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
Scanbodies | Scanbodies

As a further step into digitalization of the **BENEFIT®** System, these scanbodies were developed in cooperation with Orthodontist **Simon Graf** (Belp, Switzerland) to ensure visualization of a detailed rendering. The Scanbody Standard is used right after implant insertion. The Scanbody mini can be screwed onto the implants even after several months of treatment.

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Developed in cooperation with: | Entwickelt in Zusammenarbeit mit: Simon Graf, Orthodontist, Belp, Switzerland

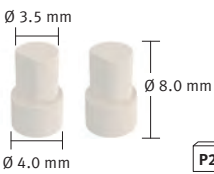Scanbody PEEK Standard Scanbody PEEK Standard	
33-10760	P2
	

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HYBRID Scanbody PEEK HYBRID Scanbody PEEK	
33-10765	P2
	

INFORMATION

In Order to prevent overload, we recommend the use of 33-18266 (see page 27).

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e.g. Screwdriver insert
z. B. Eindrehinstrument
33-54512

Illustrations are for illustrative purposes only and are not to scale, except where scale is indicated.

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Sterile Packaging | Sterile Verpackung

Implants – in single sterile packaging

The outer and inner packaging contains labels indicating UDI code – like article number, LOT number, expiration date, etc. for the patient folder and for full traceability.

The sterile screws are contained in a small colour-coded screw carrier which protects the screw and at the same time acts as a pick up tray. This carrier is placed in a clear, sealed blister package. The screw can easily be removed from the carrier by means of the self retaining screwdriver.

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The instruments can be used for several **PSM Medical** products. In order to find the right instruments for your use, they are indicated in color.

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Instruments
Instrumente

Drills | Bohrer

The instruments can be used for several **psm** products. In order to find the right instruments for your use, they are indicated in color.

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

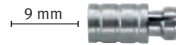
Drill Stop | Bohrerstop

Drill stop sleeve, 7 mm Bohrer-Stophülse, 7 mm		
33-18307		
Drill stop sleeve, 9 mm Bohrer-Stophülse, 9 mm		
33-18309		

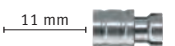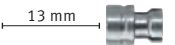Drill stop sleeve, 11 mm Bohrer-Stophülse, 11 mm		
33-18311		
Drill stop sleeve, 13 mm Bohrer-Stophülse, 13 mm		
33-18313		


Drill-Stop sleeves can be used with drills: | Bohrer-Stophülsen verwendbar mit Bohrer: **12-12625, 11-18452 & 10-67513**

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The drill stop prevents the clinician from unintentionally drilling too deep. They assure an accurate working length from 7 to 13 mm in 2 mm increments. By sliding the stop sleeve over the appropriate **PSM** drill, the working length is adjusted to the desired depth. Removal of the sleeve is made easy by using the buffer.

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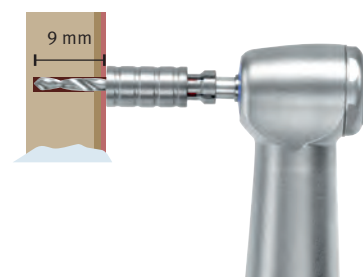
Handles | Handgriffe

The manually turned unit is connected to your existing contra-angled handpiece (blue 1:1) that enables its use like an angled screwdriver.

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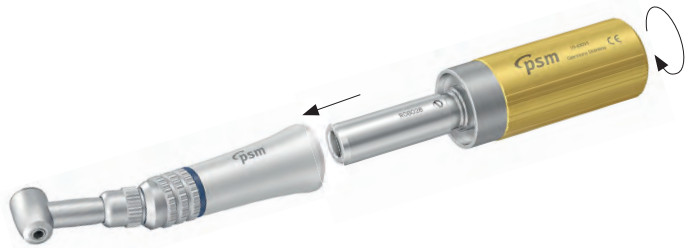10-63025

P1

Manually turned unit mod. to Pauls, with adjustable torque from 0 – 40 Ncm

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33-54704

P1

Multi purpose handle for the use with instruments with dental shaft end

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33-18236

P1

Thumb screw for Dental mandrel with limited torque (10 Ncm)

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Battery Screwdriver – iSD900 | Akku Schraubendreher iSD900

Cordless Battery Screwdriver with Torque Calibration System for the controlled fixation of PSM Mini screws.

The iSD 900 will guarantee precise torque control due to the unique – easy to use – torque calibration system (10 – 40 Ncm). Time-saving and precise, riskfree insertion and tightening of screws and abutments. The complete set contains a reliable and user friendly handpiece for Dental attachments. Reverse mode is indicated with sound. Standard rechargeable AAA Ni-Mh batteries can be used.

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median

paramedian

The soft tissue probes are infinitely adjustable and are fixed by means of the normal **BENEFIT®** instruments (i. e. **33-18288**) with set screws in the desired position.

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33-18450

The screw location probe acc. to Dr. Schlichter identifies the target points for the median or paramedian placement of the **BENEFIT®** Screws. The precise positioning of the **BENEFIT®** Implants improves the location and parallelism with each other. It also increases stability and simplifies the chairside bending of the plate. Additionally the lab time is reduced due to less bending.

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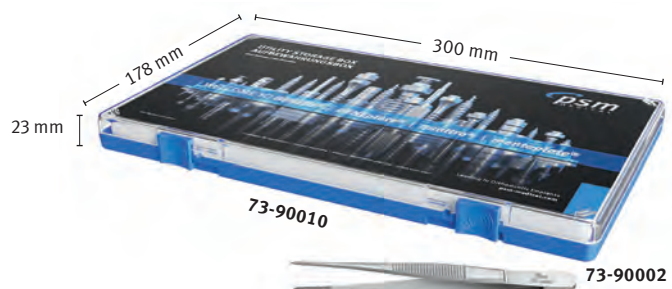
73-31990

BENEFIT® Utility Storage Tray Set,
consisting of 73-90010 and 73-90002

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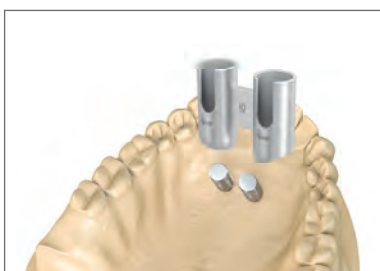
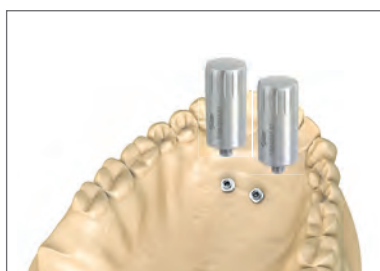
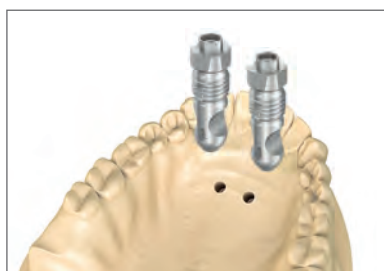
The matching drill (**33-10914**) and screwholder with stop (**33-10903**) you will find on page 24.

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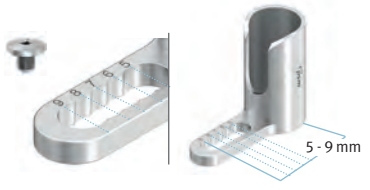
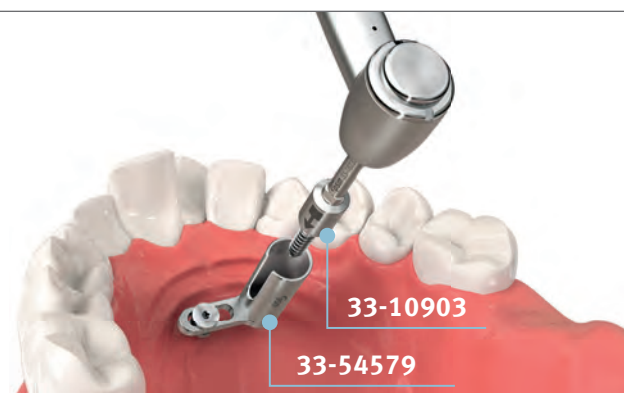
Discover new standards for your orthodontic care –
from precision implant screws to intelligently designed system components. Take a look at our catalogues and see for yourself the variety and quality of our products.

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Distributor Search

Notes | Notizen

Specifications and design are subject to change without prior notice. All pictures, photos and product descriptions are the property of PSM Medical GmbH. Utilization and copies by third parties have to be authorized by PSM Medical GmbH. All rights reserved! All trademarks and brands mentioned in this brochure are subject to unrestricted regulations of the trade mark laws valid and right of ownership of the registered owner.

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PSM North America, Inc.